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- Anleitung -

Seite 1 von 2

Version OM07-007a_v1.1 gültig ab 19-Feb-2025

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EDTA blood collection and shipment

- Instructions -

Page 2 from 2

Version OM07-007a_v1.1 valid from 19-Feb-2025

Omegamatrix Method / HS-Omega-3 Index® **Collection and shipment of EDTA-blood**

Please read these instructions completely and carefully before you start. These instructions describe the procedure for collecting and shipping an EDTA blood sample for the analysis of red blood cells using the Omegamatrix Method. It accurately measures 26 fatty acids including the HS-Omega-3 Index® and HS-Trans-Index® (see www.omegamatrix.eu for more information).

How it works:

Patient and sender data

- Please enter the patient's details (name, address, date of birth, gender) legibly in the left-hand field of the referral form.
- Enter the sender's details (name, address) legibly in the right-hand field of the referral form.
- By default, the reports are sent to the sender and the invoice is sent to the patient. If you would prefer a different arrangement, please specify this on the form.
- By signing the referral form, the patient confirms the order and their acknowledgment of the data protection declaration.

EDTA-blood sample

- We require a tube (approx. 2 ml) of EDTA blood for the analysis. The patient **does not have to fast** before collecting the sample.
- Please label the sample so that it can be clearly assigned to the patient (e.g. name and date of birth) and indicate the date of blood collection.
- Place the sample in the blue box and send it together with the referral form in the stamped envelope by normal post **on the day of collection** (!). The sample is usually stable for 7 - 10 days.

What next?

- We usually begin the analysis on the day the sample arrives.
- The processing time is approximately 2 working days.
- After the analysis is complete, you can access the detailed report via our laboratory portal at lab.omegamatrix.eu by entering your e-mail address.

If you have any questions about the reports, you can reach us by phone on +49 89 55 06 30 07 or via e-mail at ticket@omegamatrix.eu. For orders, please contact us at orders@omegamatrix.eu.